



NIPT

Non-Invasive Prenatal Test

SAMPLING INSTRUCTIONS

1. The blood can be collected from week 11 of the pregnancy
2. 1 full STRECK tube from the mother is needed
3. Indicate the code and date of birth of the mother on the blood tube
4. Fill the blood tube FULLY and invert the tube five times
5. Insert the blood tube back into the plastic holder
6. Complete the application form and sign it
7. Put the plastic holder with the blood tube and the application form in the plastic bag
8. Send everything at room temperature (do not freeze or place in the refrigerator) with express mail (FEDEX) with the GENDIA FEDEX customer number 274754214

For questions or information, contact GENDIA

By e-mail : NIPT@GENDIA.net

By phone: + 32 (0)3 303 08 01